

Release Note 3.45.3.2956



1 PURPOSE

This document describes the Release Notes for the Spirare Software, including Changes made to the SW, bug fixes, release version and date and known issues.

2 DOCUMENT CONTENT

2.1 Version

The version number for this release is 3.45.3.2956.

New version is release due to add bacterial filter correction and minor bug fixes.

2.2 Changes

Spirometry

- Viral/bacterial filter correction when conducting an inspiration test. The system will ask you before starting an inspiration test if you are using a viral/bacterial filter. You may also mark previous tests that have been conducted with a viral/bacterial filter.

2.3 Bug fixes

Spirometry

- Tidal volume. FVC was calculated wrong for children since it is difficult to establish the start of test.

ECG

- Broken Lead Detection was always red even when the ECG signal was correct.

Various

- Smoking status was deactivated in journal when finishing a spirometry test.
- Some of the configuration setup was enabled for non-administrative users.
- SPShell module could not connect to the database.
- Blank page when printing out from Citrix environment.

2.4 Known issues

- It is possible to conduct an “FVC with medication” without running a pre-test, using the menu under Spirometry -> Select test. The graph is missing when conducting this test.
- The build-in system for asking about 14 days temporary license renewal does not work.
- When using tidal volume mode, there is no sound beep at the end of the forced expiration.

- Quality Control shows wrong parameters for Inspiration.

2.5 Release date

Release date for this version is 12-05-2023.